











AquaSplint® Thermo und Mini –

self-adjusting and customizable CMD occlusal splint

Advantages compared to classical occlusal splints

 <p>Fast pain relief and muscle relaxation</p>	 <p>Can be used immediately without lab works</p>	 <p>Universal use due to universal size</p>	 <p>Custom-fit and comfortable seating</p>
 <p>Wearing time of 4 weeks / 10 h per day is sufficient</p>	 <p>No adjustment appointments (thanks to the hydrostatic balance of the waterfilled pads)</p>	 <p>Replaces relaxation and distraction splints</p>	 <p>Can be used during orthodontic treatment</p>



AquaSplint® Thermo and Mini are self-adjusting and customizable ready-made CMD splints which can be inserted without lab works, impressions or registrations in only a few minutes. They are made up of two water-filled pads connected by a tube (Hydrostatic Aqua-Balance).



Can be used during an orthodontic treatment (brackets / aligners)

1. AquaSplint® Thermo with brackets
2. AquaSplint® Thermo with aligners
3. AquaSplint® Mini with brackets



The difference between AquaSplint® Thermo and Mini

AquaSplint® Thermo	AquaSplint® Mini
Height: 3 mm: optimal for patients with deep bite	Height 1.5 mm: optimal for patients with edge to edge / open bite
Is customized through the lining underneath the saddle	Does not have a saddle / lining and is customized via a steel arch in the tube

Indications:

1. Fast pain relief and muscle relaxation
2. Optimal initial treatment of CMD
3. Differential diagnosis of multifactorial origins of CMD
4. Supporting therapy of diseases such as cervical spine syndrome, tinnitus, migraine, stress, trauma, surgery, limitations and impairments (CMD has a multifactorial origin – the malocclusion is not always the main cause)
5. Simple pre-prosthetic / pre-orthodontic bite identification
6. Neutralization of functional disorders, premature contacts or crossbites with dental compensation

Contraindications:

1. Severe Bruxism / grinding - as it can reduce the durability
2. Sharp edges on prosthetics, crowns or others can damage the water-filled pads (a thermoformed splint with a thickness of 0.5 mm on the opposing jaw can help)
3. Missing posterior teeth – at least one second premolar has to be present in each quadrant

Order details

Order No.	Content	Description
406-0001	1	AquaSplint® Mini
406-0002	1	AquaSplint® Thermo

Inventor:

Dr. Aladin Sabbagh



Manufacturer:

PLASTYLE GmbH
Lessingstraße 5
91080 Uttenreuth
Phone: +49 (0)9131 9402406
info@plastyle.de

Distributed by:

FORESTADENT

Bernhard Förster GmbH
Westliche Karl-Friedrich-Str. 151
75172 Pforzheim (Germany)
Phone: +49 (0) 7231 459-0
Fax: +49 (0) 7231 459-102
www.forestadent.com
info@forestadent.com

 **FORESTADENT®**
GERMAN PRECISION IN ORTHODONTICS