**GB** **Frog Appliance\***  
Delivery instructions

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Phone +49 (0) 7231 459-0 · Fax +49 (0) 7231 459-102

info@forestadent.com · www.forestadent.com

  
**FORESTADENT<sup>®</sup>**  
GERMAN PRECISION IN ORTHODONTICS

1

**GB**

- a) Thoroughly clean, etch, and rinse the anchor teeth.
- b) Insert the ends of the distalizing spring into the lingual sheaths of the molar bands and press the Nance Button against the palate.

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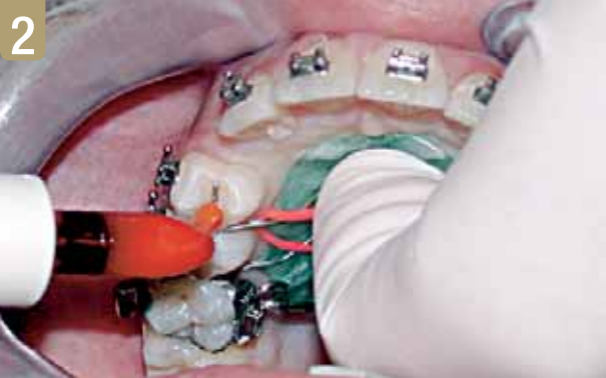
2

**GB**

- c) Paint unfilled composite resin on the etched surfaces of the anchor teeth.
- d) Place filled composite resin over the anchor wires.

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3

**GB**

e) With finger pressure hold the appliance firmly in place against the palate and light cure the composite resin.

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4

**GB**

- f) Remove the securing elastic and activate the appliance. **NOTE:** One complete rotation around own axis of the activation screw opens the appliance 0.4 mm. Three rotations are recommended for 4 to 5 week intervals and 5 rotations are recommended for 8-week intervals. If second molars are erupted 3 revolutions with 5 to 6 week intervals are recommended. More than 3 rotations are not recommended when second molars are erupted.

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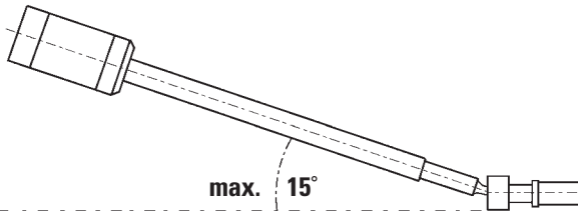
5

**GB**

- g) **Caution! Only use the original screw driver that is delivered with the FROG! Don't use any other screw driver!** Make sure the angle of the screw driver to the FROG screw does not exceed  $15^\circ$  when placed inside the FROG screw for activation. The key needs to be properly inserted into the screw head.

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