




AquaSplint® Mini

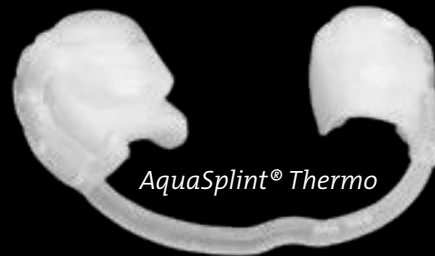

AquaSplint® Thermo



AquaSplint® Thermo and Mini –
self-adjusting and customizable CMD occlusal splint

www.forestadent.com

FORESTADENT®
GERMAN PRECISION IN ORTHODONTICS



Can be used during an orthodontic treatment (brackets / aligners):




AquaSplint® Thermo with brackets



AquaSplint® Thermo with aligners



AquaSplint® Mini with brackets

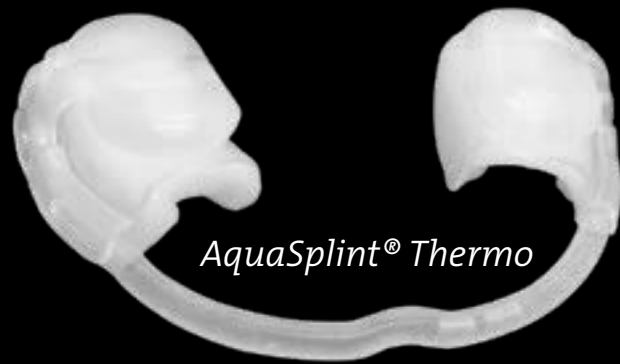

AquaSplint® Thermo



AquaSplint® Mini

AquaSplint® Thermo and Mini are self-adjusting and customizable ready-made CMD splints which can be inserted without lab works, impressions or registrations in only a few minutes. They are made up of two water-filled pads connected by a tube (Hydrostatic Aqua-Balance).

*The difference between
AquaSplint® Thermo & Mini:*

AquaSplint® Thermo	AquaSplint® Mini
Height: 3 mm: optimal for patients with deep bite	Height 1.5 mm: optimal for patients with edge to edge / open bite
Is customized through the lining underneath the saddle	Does not have a saddle/lining and is customized via a steel arch in the tube

Indications:

1. *Fast pain relief and muscle relaxation*
2. *Optimal initial treatment of CMD*
3. *Differential diagnosis of multifactorial origins of CMD*
4. *Supporting therapy of diseases such as cervical spine syndrome, tinnitus, migraine, stress, trauma, surgery, limitations and impairments (CMD has a multifactorial origin – the malocclusion is not always the main cause)*
5. *Simple pre-prosthetic / pre-orthodontic bite identification*
6. *Neutralization of functional disorders, premature contacts or crossbites with dental compensation*

Contraindications:

1. *Severe Bruxism / grinding - as it can reduce the durability*
2. *Sharp edges on prosthetics, crowns or others can damage the water-filled pads (a thermoformed splint with a thickness of 0.5 mm on the opposing jaw can help)*
3. *Missing posterior teeth – at least one second premolar has to be present in each quadrant*



Advantages compared to classical occlusal splints:




Fast pain relief and muscle relaxation



Can be used immediately without lab works



Universal use due to universal size

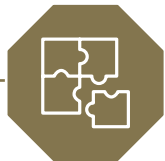

Custom-fit and comfortable seating



No sharp edges / pressure marks



Wearing time of 4 weeks / 10 h per day is sufficient



No adjustment appointments (thanks to the hydrostatic balance of the water-filled pads)



Replaces relaxation and distraction splints



Can be used during orthodontic treatment



Order details

Order No.		Description	Content
406-0002		AquaSplint® Thermo	1
406-0001		AquaSplint® Mini	1





Inventor:

Dr. Aladin Sabbagh



Manufacturer:

*PLASTYLE GmbH
Lessingstraße 5
91080 Uttenreuth (Germany)
Phone: +49 (0)9131 9402406
info@plastyle.de*

Distributed by:

FORESTADENT

*Bernhard Förster GmbH
Westliche Karl-Friedrich-Straße 151
75172 Pforzheim (Germany)
Phone: +49 (0) 7231 459-0
Fax: +49 (0) 7231 459-102
www.forestadent.com
info@forestadent.com*

 **FORESTADENT®**
GERMAN PRECISION IN ORTHODONTICS