

Necessary information to generate an Accusmile® treatment proposal

In order to provide the best possible treatment plan for your patient we need the following case-relevant information from you:


1. Photographs with three-dimensional head positioning



2. Photograph from the front with smile




3. Panoramic x-ray and lateral ceph in perfect image quality and without positioning flaws during recording. The lateral ceph should give a clear image of the soft tissue contour.

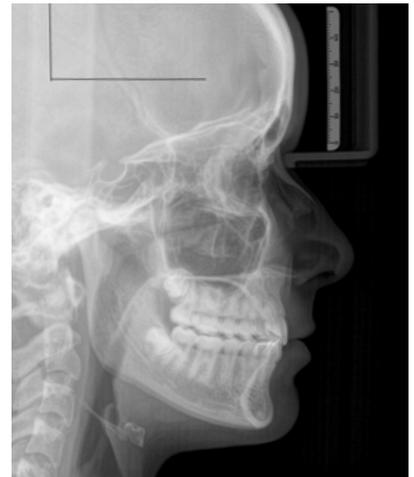

4. Orthodontic plaster model or alternatively 3D scan files

With plaster models the valid norms have to be met. Next to the teeth the maxillary tuberosity, the alveolar processes and ligaments have to be clearly visible. Please pay attention to a precise three-dimensional orientation when trimming. Otherwise misinterpretation of the occlusion might occur.

Important: In order to generate a treatment proposal both jaws and the bite registration are mandatory.

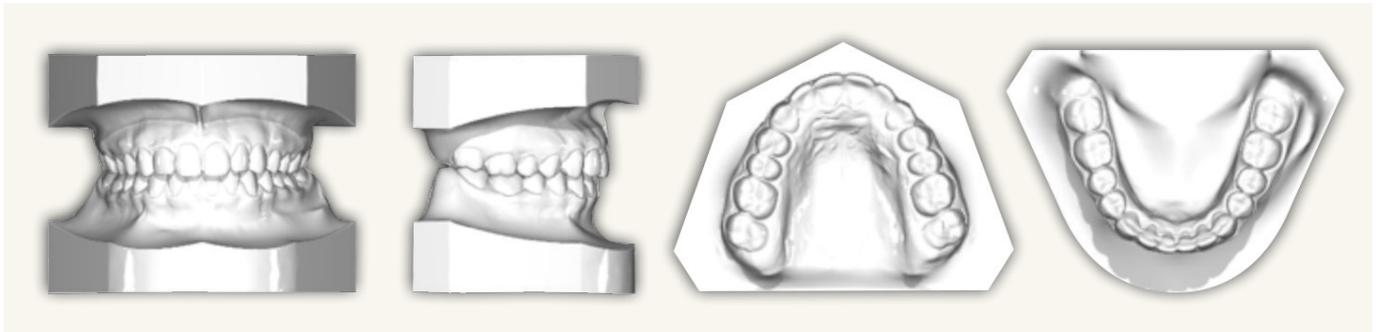

5. Intra-oral photos

The left and right lateral photos are especially important. Ideally these pictures are taken using a dental photo mirror. The focus should be placed on the second premolars. For high quality intra-oral pictures we recommend the use of a SLR with ring flash.

