OrthoEasy®

UNSTERIL · NON-STERILE
NO ESTÉRIL · NON STÉRILE

Gebrauchsanweisung
Instructions
Instrucciones
Instructions

CE 0297 93/42/EWG

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OrthoEasy® – Non-sterile

Purpose

FORESTADENT OrthoEasy® miniscrews (Pins and Pal) serve as temporary fixed anchorage points for orthodontic appliances during treatment of malocclusions of teeth, jaws and masticatory system.

Single use

OrthoEasy® miniscrews are intended for single use. Screws that have already been used must not be used again under any circumstances. Removing the screws, reprocessing them and using them again may alter the product properties, such as dimensional stability and shape retention, to an unforeseeable extent.

Performance characteristics and specifications

Feature	Size
Length	See "Types and accessories"
Thread properties	Self-tapping, self-drilling
Outer thread diameter of the implantable thread section	1.7 mm
Fracture torque	≥ 30 Ncm

Insertion torque	20 Ncm (see also the "Insertion procedure" section)
Delivery condition	NON-sterile
OrthoEasy® Pin	
Cross slot	0.022
OrthoEasy® Pal	
Can only work with an abutment or fixation plate and retaining screw (see "Accessories")	

Storage

OrthoEasy® miniscrews must be stored in a dry place protected from dust and direct sunlight and at room temperature.

Indication

The need for a firmly fixed (absolute) anchorage as part of orthodontic treatment.

Contraindication

OrthoEasy® Pins and Pal must not be used under the following conditions:

- Poor bone quality
- Osteoporosis, osteolysis, osteomyelitis
- Impaired revascularisation or infection can result in the product detaching, bending, splitting or breaking or prematurely coming loose from the bone
- Sensitivity to titanium or allergic reactions to foreign bodies

Patient groups

Adolescents and adults requiring orthodontic treatment.

User and Requirements for users

OrthoEasy® miniscrews may only be used by orthodontists, dentists and clinicians who are familiar with dental surgery, including diagnosis and preoperative planning. This also includes knowledge of indications and general rules for hygiene, dental/surgical conduct and compliance with regulations on occupational health and safety and accident prevention.

For clinicians who are inexperienced in implant procedures, these instructions on their own are insufficient for their correct application. Refrain from using this equipment if there are any ambiguities about the topics mentioned and do not start until these have been fully resolved. In general, we recommend that clinicians are instructed by an experienced user or that they attend one of our training courses.

DE

EN

ES

FR

As the application of the product lies outside the control of the manufacturer, any liability for damage caused by such use shall be excluded. Responsibility shall rest solely with the clinician.

SSCP

The Summary of Safety and Clinical Performance (SSCP) is available in the European database on medical devices (Eudamed; <https://ec.europa.eu/tools/eudamed/eudamed>), where it is linked to the Basic UDI-DI (++EFOR100101CZ). In the transition phase until the Eudamed is fully functional, the SSCP can be requested from the manufacturer.

Residual risks and side effects

In order to avoid the following residual risks, we recommend using an insertion guide (e.g. FORESTADENT Accuguide), particularly when

using OrthoEasy® Pal:

- Tissue contusion
- Perforation of the nasal and maxillary sinus membrane

Additional residual risks and side effects include:

- Damage to roots if mini implants are positioned too close to them
- Plaque, gingivitis and other inflammations caused by poor oral hygiene

Preparation

Non-sterile OrthoEasy® miniscrews must be sterilised before use. Likewise, the instruments used for implantation must be cleaned and sterilised. To do this, the products must be taken out of their packaging and put into a container suitable for sterilisation (e.g. Order No. 1199-0000 – OrthoEasy® rack).

Sterilisation of the OrthoEasy® miniscrews

Please refer to the instructions of the sterilisation system manufacturer with respect to the duration, temperature, pressure and post-drying time for sterilisation.

The following parameters have been validated for the fractionated steam sterilisation method:

- Temperature: 133–135 °C (270–275 °F) 3 x pre-vacuum.
- Standard cycle: 15 minutes (min. 6 minutes).
- Post-drying time: 8 minutes.

FORESTADENT shall assume no responsibility for the user's correct implementation of sterilisation.

Notes on use

OrthoEasy® miniscrews must only be used with original instruments and accessories from FORESTADENT. These were tailored to each oth-

er during development and production. Any use of products from other manufacturers may lead to unforeseeable risks and is therefore not recommended. For further information, please do not hesitate to contact us or our authorised retailers. In order to correctly set and align the **OrthoEasy® Pins**, users will require an octagonal blade from the OrthoEasy® range of accessories. The octagonal blade comes with a red ring marking. The head of the octagonal blade has two black laser markings which can be lined up with the slot of the OrthoEasy® Pins. This indicates the slot's position to the user during the insertion of the pin (see image).

OrthoEasy® Pal miniscrews must be used in pairs because this is the only way of protecting them from torsional strain.

The OrthoEasy® Pal retaining screws have a lock which holds the screw on the blade for the OrthoEasy® Pal retaining screw. In order to ensure that this lock works properly, the OrthoEasy® Pal retaining screws must only be used once. The integrated locking function of the OrthoEasy® Pal retaining screws can result in the blade for the OrthoEasy® Pal retaining screw canting in the screw after it is screwed in at 10 Ncm. The blade

can easily be detached again by carefully moving it back and forth. For this reason, the blade for the OrthoEasy® Pal retaining screw is subject to a certain amount of wear. As the user, you should therefore always check whether the retaining screw is still firmly secured on the blade before use. If this is not the case, the retaining screw should be replaced immediately.

Regularly check whether the OrthoEasy® Pal retaining screws have become loose and, if necessary, retighten them to 10 Ncm.

Insertion sites

OrthoEasy® Pins can be inserted in the vestibule or palate of the maxilla or on the alveolar ridge. In the mandible, they should only be inserted in the vestibule at the centre of the alveolar ridge, as a general rule. The Pins must not be in-

DE

EN

ES

FR

served in the area of the movable mucosa, on the lingual side of the mandible or in the vicinity of extraction wounds or dental follicles.

The **OrthoEasy® Pal** has been specially designed for palatal use and is used in pairs. The amount of bone must be sufficient for an insertion length of 8 mm in order to prevent perforation of the opposite side. The "T zone" is suitable for inserting the screws.

The choice of miniscrew and insertion site must be based on X-ray guidance. The amount of bone, planned load, position of nerves and vessels, and the patient's dental structures must be taken into due consideration. If an excessive amount of force is applied in relation to the size of the screws, this may result in the screw being lost or broken. The length of the screw must be chosen such that perforation of the opposite side is excluded.

Insertion procedure

- Choose the insertion site (see above), ensuring that the subsequent load is at right angles to the longitudinal axis of the implant, to the extent possible.
- Carefully clean, disinfect and sterilise all of the instruments required for the procedure.
- Administer local anaesthetic.
- Put the screw in the insertion instrument.
- Screw into the bone in a smooth movement and avoid canting. The maximum torque is 20 Ncm. In the case of automated insertion, the maximum torque should be preset and the speed should be limited to between 30 and 50 rpm. Continuously irrigate the OrthoEasy® miniscrew with sterile saline solution during the entire insertion process (whether performed manually or automatically). This is used for

cooling and prevents deposits from forming. Screw in the miniscrew as fully as possible without contusing the tissue below the head.

- As a rule, the OrthoEasy® miniscrews can be loaded immediately if there is sufficient primary stability. The maximum load is 2 N (approximately 200 g). At the start of treatment, it is recommended that clinicians limit the application of force to approx. 1 N (about 100 g) for two months.

OrthoEasy® Pins

- Immediate loading of OrthoEasy® Pins: Insert a connecting element into the implant's slot used to connect to orthodontic elements (brackets, bands etc.) or position the coupling elements and secure them using a light-curing bonding agent.
- If the OrthoEasy® Pins are not immediately

included into the treatment, they should be covered to ensure better comfort and hygiene. OrthoEasy® Pins can either be fitted with rubber rings (e.g. elastic separators, Order No. 657-0002) or you can apply some light-curing bonding agent and chamfer them.

OrthoEasy® Pal

- When inserting the OrthoEasy® Pal, we recommend using an insertion guide (e.g. FORESTADENT Accuguide).
- Immediate loading of OrthoEasy® Pal: Position the appliance welded to the abutment and secure it with OrthoEasy® Pal retaining screws. Maximum torque 10 Ncm.
- If the OrthoEasy® Pal miniscrews are not immediately included in the treatment, they should be covered to ensure better comfort and hygiene.

In the case of the OrthoEasy® Pal, the abutment performs this function and must be secured with OrthoEasy® Pal retaining screws.

Precautions during insertion

- Protect your patient. Secure miniscrews and small instruments during intraoral use to prevent patients from aspirating or swallowing them.
- Check instruments, appliances, fastening elements and miniscrews for damage before use.
- Damaged or non-sterile products must not be used for treatment.
- Only insert and remove the OrthoEasy® miniscrews using the instruments intended for these purposes. Using excessive force when fastening the screw can cause it to be unstable. In the case of automatic insertion, please adhere to the torque limits and speeds.

- When sterile screws are removed from their sterile packaging, ensure that they do not come into contact with the lips of your patient or are contaminated in any other way.
- If you have any technical questions or require further product information, please contact your retailer or the manufacturer.

Removal procedure

- Administer local anaesthetic.
- Remove the connected appliances.
- Unscrew the miniscrew manually with the handle or automatically (setting the torque and speed in the same way as for insertion).

Patient information

Every patient must be informed about the planned procedure in accordance with the legal require-

DE

EN

ES

FR

ments. Informing patients about hygiene and the technique are key to the success of the treatment. Surgical information must refer to potential risks, such as damage to roots or nerves, bleeding, pain and the premature loss of miniscrews.

You should also inform the patient that improper manipulation of the miniscrew or appliance could lead to them coming loose or being lost and tell them what course of action they should take in these instances.

Patients must keep to the check-ups scheduled by the attending clinician.

Should there be any side effects, changes or pain, the attending clinician must be informed immediately.

Material

OrthoEasy® miniscrews are made of Ti-6Al-4V

(Ti Grade 5, ASTM F136 DIN EN ISO 5832-3), a material approved for implant dentistry.

Disposal/recycling

The screws must be disposed of in compliance with the regional or national legal requirements for materials used in clinical practice.

Notes on the obligation to report

The manufacturer and the responsible national authority must be informed of any serious adverse events associated with the product.

Implant passport

Please use the implant passport included in the scope of delivery and the individually enclosed patient labels in order to provide your patient with the relevant information. To do this, apply the pa-

tient label to a corresponding field in the implant passport and add the date. Complete the fields provided with your patient's name and address, and also the address of your practice.

Types and accessories

OrthoEasy® Pins are available in different shank lengths that are colour-coded with a titanium oxide coating. The head of the OrthoEasy® Pins is equipped with a cross slot for attaching orthodontic appliances. The geometry of the head has been designed for safe use with the original accessories from our range.

The **OrthoEasy® Pal** differs from other OrthoEasy® miniscrews in the colour of its titanium oxide coating and its head. The head of the OrthoEasy® Pal includes a snug fit for the OrthoEasy® Pal abutment and an inner thread

(M1.7 x 2.2) for securing the abutment using a retaining screw. The geometry of the head has been designed for safe use with the original accessories from our range of accessories.

Types	Order No.	Colour
OrthoEasy® Pin .022"/1.7 x 6 mm	1101A2306	Rose
OrthoEasy® Pin .022"/1.7 x 8 mm	1101A2308	Pink
OrthoEasy® Pin .022"/1.7 x 10 mm	1101A2310	Gold
OrthoEasy® Pal 1.7 x 8 mm	1201A2308	Green

OrthoEasy® Pal		
OrthoEasy® Pal abutment	1299-1004	N/A
OrthoEasy® Pal abutment, high profile	1299-1014	N/A
OrthoEasy® Pal retaining screw	1299-1003	N/A

OrthoEasy® miniscrews must only be used with original instruments and accessories from FORESTADENT (see "Notes on use").

Accessories	Order No.
General accessories	
OrthoEasy® rack For steam sterilization of pins, instruments, etc.	1199-0000
OrthoEasy® manual drive attachment For manual insertion with contra- angle handpiece	1299-1007
OrthoEasy® octagonal For screwdriver	1199-0006 1199-0015
OrthoEasy® octagonal blade For use in contra-angled handpiece	1199-0007 1199-0008

DE

EN

ES

FR

OrthoEasy® Pin	
OrthoEasy® handle for screw-drivers Handle for blades	1199-0002
Memory Titanol-Feder .018" x .025" (Slot .022") For molar-uprighting	307-1012
Memory Titanol-Feder .016" x .022" (Slot .018") For molar-uprighting	307-1010
Space Jet For space closure	307-1070

OrthoEasy® Pal	
Blade for OrthoEasy® Pal retaining screw	1299-1006
SmartDrive torque driver with torque control For blade for retaining screw	1299-0007
OrthoEasy® Pal lab analog For preparation of the application in the laboratory	1299-1001
OrthoEasy® Pal impression cap (lab) For conventional impressions (Silicone, alginate)	1299-1005

Fixation plate with parallel wire (ø 1,1 mm) For appliances for mesialization or distalization of molars in the maxilla	1299-0100
Fixation plate with orthogonal wire (ø 1,1 mm) For appliances for mesialization or distalization of molars in the maxilla	1299-0101
Fixation plate with wire in T-shape (ø 1,1 mm) For appliances for mesialization or distalization of molars in the maxilla	1299-0102
SmartJet™ Tube, Distal-Mesial Slider For appliances for mesialization or distalization of molars in the maxilla	322-0110

Cleaning, disinfection and sterilisation of OrthoEasy® accessories

Automated disinfection, alkaline cleaning/disinfection, with ultrasound treatment if necessary (e.g. pre-rinse, clean with alkaline agents (pH > 10), at a high temperature of min. 55 °C, intermediate rinsing and then thermal disinfection/after-rinsing).

The accessories can be sterilised under the same conditions as those described for miniscrews under "Preparation".

(You can find detailed preparation instructions on our website.)

Labelling and symbols

Observe the instructions for use

Do not reuse

Observe the accompanying documents

Order No.

Batch number

Date of manufacture

Manufacturer

Expires

R_x only For use by trained personnel only

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Bernhard Förster GmbH